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**ASSINGMENT:**

Foot and Shin brace is a product with wide range of appliance. Effectively provides support for stable fracture of foot and/ or ankle. Provides stabilization in soft tissue injuries, and during post-operative phase of rehabilitation. Construction provides a very good stability and allows angle flexion adjustments both plantar and dorsal. Appropriate adjustment of the orthosis is provided by five seperated straps.

**INDICATIONS:**

- Fracture and dislocation of the metatarsal bones.
- Acute conditions and during post-operative treatment.
- Rapture of the tarsal bones and calcaneus.
- Surgery on the joint of the big toe.
- Soft tissue injury.

- Rapture of the Achilles tendon.
- Severe ankle sprains.
- Conservative treatment after surgery.

**CONTRAINDICATIONS:**

- Conditions which require cast immobilization.
- Long-healing wounds and ulcers.

**PRODUCT ADJUSTMENT:**

1. Remove the TD ROM Walker from the large protective plastic bag and detach the foam bootie. DO NOT remove the plastic sleeves that are over the uprights. These plastic sleeves must remain over the uprights during the initial phases of the - fitting procedure.
2. Open the leg and foot sections of the foam bootie by releasing the Velcro hook from the foam. Have the patient slide their foot and lower leg into the bootie.
3. Close the bootie, making sure that the foam is wrapped snugly around the patient's limb. The pre-sized bootie is designed with sewn edges, and you should not have to trim it to size (it can be cut, however, if needed).
4. To adjust the ROM stops, use the 1/4 Allen key provided with the walker and remove the Allen bolt. Rotate the acrylic cap to the position where the flexion/extension lock pins are set. Reposition the pins in the desired holes, rotate the cap to its original position, reinsert the Allen bolt and tighten it securely with the Allen key.
5. Make sure that the plastic sleeves over the uprights are still completely covering the Velcro hook that is attached to the inside of the uprights. Have the patient place his or her foot in the aluminum shell between the two uprights.
6. To optimize stabilization, the uprights can be contoured to ensure an intimate alignment with the sides of the calf. There are also foam comfort pads mounted on the inside of the uprights (near the ankles) that can be moved to enhance comfort, or that can be removed to increase the medial-lateral width between the uprights.
7. When you are sure the uprights are properly contoured and in the proper alignment (centered at midline on the sides of the leg), remove the plastic sleeves from the uprights. The Velcro hook on the inside of the uprights will lock onto the foam bootie.
8. Tighten the two forefoot straps that secure the patient's foot in the boot. The forefoot straps should fit snugly. These straps can be adjusted by simply loosening or tightening the Velcro hook tabs. Any excessive material can also be trimmed off for a cleaner look.
9. Tighten the three straps that secure the lower limb between the uprights, starting with the strap closest to the ankle. These

straps are in alternating directions to prevent rotation. The three straps should be tightened firmly.

10. To increase compression, the pneumatic bladder can be inflated by compressing the ball pump on the front of the bootie. To deflate the pneumatic bladder (to reduce compression), PUSH IN and hold the black valve located directly below the ball pump.

**ATTENTION!**

The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients to attempt to apply or adjust this device unless otherwise instructed to do by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional. as they know the individual condition for which this device was prescribed.

**CLEANING INSTRUCTIONS:**

Remove the inner pad from the brace. Rinse the inner pad in water not exceeding 35° with mild detergent for about 5 minutes. Line dry in the shade. Keep out of full sun.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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