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## INSTRUCTIONS FOR USE

### SPINCARE PIN - Lumbosacral orthosis

#### INDICATIONS:

- Nonoperative treatment of spinal segment dysfunctions of different etiology.
- Stabilization of lumbar area in case of segmental instability after surgeries.
- Follow- up treatment after neurosurgical/orthopedic interventions.
- Prophylaxis against injuries, overload.
- Congenital defects of lumbar area (with or without pain syndromes).
- Degenerative spinal disorders with pain syndrome.

#### CONTRAINDICATIONS:

- Conditions requiring a full stiffening. This product is not intended to be used directly on open wounds.

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1. Select the appropriate size of the orthosis on the basis of the information contained in Table sizes.
2. Release the orthosis and adjust it to the body.
3. Make sure that the orthosis is in the right place and covers the lumbar-sacral area.
4. Fasten the orthosis in front of as accurately as possible.
5. Adjust compression using fasteners. After obtaining the desired compression pin fastening to the orthosis.

#### SIZING:

Measure waist circumference to select proper size.

| Rozmiar                  | S       | M       | L       | XL       | XXL       | XXXL      |
|--------------------------|---------|---------|---------|----------|-----------|-----------|
| Waist circumference (cm) | 56 – 71 | 69 - 84 | 81 - 96 | 94 - 109 | 107 - 122 | 120 - 135 |
| Height (cm)              | 25      |         |         |          |           |           |

#### KONSERWACJA:

Before laundering, remove the plastic inserts and close all fasteners before washing. Orthosis should be washed by hand using a mild detergent in cold water. Do not dry clean and ironed.

#### WARNING!

Do not wear brace directly on skin.