PROHAND

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This product complies with the EC directive 93/42/EEC for medical devices.

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PROHAND wrist support provides optimal wrist joint stabilization, prophylaxis of injuries and overstrain. It effectively assists follow-up treatment and protects wrist while working, practicing sport, and during other activities which generate significant load. High-quality fabric ideally adapts to your body and ensures comfort of use.

Thank you very much for selecting a product of the Meyra-Ortopedia Group. The product complies with all legal regulations according to the Medical Products Law.

Please read the directions for use carefully and keep to the advice and recommendations given. Should you have further questions please address the responsible physician, your local specialist dealer or our service hotline.

Product description

Elastic, breathable fabric provides optimal wrist compression, stabilization and prevents oedema.

Prohand is available in one universal version, suitable for right and left hand. Due to simple construction it is easy to use and maintains proper medical and functional characteristics.

Safety instructions

The first application of the product must be carried out by special personnel trained on these kind of products as otherwise the product liability cannot be guaranteed. The product must not be worn on open wounds.

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Indications:

Follow-up treatment after wrist injuries.
Wrist inflammation.
Pain syndromes of different etiology.
Degenerative wrist disorders.
Wrist protection while taking up sport
and professional activities after injury.

Directions for use

Prohand should be used according to
doctor's/physiotherapist's advice.

One universal size is suitable for every
wrist circumference.

For the first time, Prohand should be put
on with the help of medical specialist who
will give you all necessary instructions.

While putting the orthosis make sure that
the thumb is placed in the thumb hold.
Then, enfold the band around the wrist.
Start from palmar part, then continue to
the outside part of the palm. Fasten with
Velcro band.

Wear time

Wear time is determined in consultation
with your doctor.

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Mechanism of acting

Construction provides optimal wrist
compression , improves joint stabilization and
relieves muscle-ligament structures. Properly
shaped thumb hold enables unhampered
movements.

Guarantee

The guarantee lies within the framework of the
legal regulations.

Disposal

The product can be disposed of with the normal
household waste.

Storage information

The storage location should be dry and at room
temperature. The product has to be protected
against direct sunlight exposure and humidity.

Cleaning information

Wash by hand using gentle detergents in
temperature max 35 ° C. Do not
chlorinate, do not dry clean, wring gently
before drying. Do not dry close to the
artificial warmth source. Do not iron.

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