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EN

Intended use/Indications:

Support for treating ankle joint injuries: first- and second-degree sprains, ligament damage. Stabilisation in cases of permanent or temporary ankle joint instability. Protects the joints from excessive eversion and inversion movements. Habitual sprain of the ankle joint. Prevention of injuries and strains in this part of the body.

Contraindications:

 Unstable fractures require stiffening in a cast. Patients with impaired blood supply to the extremities due to diabetes or peripheral vascular disease must take special care. Do not wear the brace on open wounds. Do not use in areas other than the ankle.

Intended user:

The medical device is intended for the general public (a non-professional user).

Target patient groups:

The medical device is intended for use in patients with first- and second-degree ankle sprain, ligament damage and indications for stabilisation in cases of permanent or temporary ankle joint instability.

Disposal of the medical device:

Disposal of the device must be carried out in line with the applicable local environmental regulations.

How to put on the brace:

1. Undo all fastenings before fitting the brace.
2. Lower the side stiffeners down to a horizontal position.
3. Slide the foot in from the back. The midline of the seam must coincide with the midline of the foot. (Fig. 1)
4. Pull the side (A) and hook fastening (B) in opposite

- directions. Fasten the lower hook fastening (B).
5. Tighten the side (A) again and fasten the hook fastening (C). (Fig. 2)
6. Pull the straps at the front (D and E), crossing them in opposite directions. Fasten the hook fastening to the body of the brace on opposite sides. (Fig. 3)
7. Adjust the side stiffeners in the vertical position (F). Make sure that the recesses in the guards are on the tops of the ankle bones – both on the inside and outside (G). (Fig. 4)
8. Thread the top strap through the holes in the side guards, wrapping them around the leg. (Fig. 5 and 6)
9. Thread the end of the strap through the plastic spacer and bend it in the opposite direction. (Fig. 7)
10. Hold the stiffeners with one hand and fasten the hook fastening.
11. Try putting weight on the limb and check if the fastening provides adequate stabilisation.
12. If necessary, put the brace on again for full comfort and proper stabilisation strength.
13. Rear view (I), side view (II), front view (III). (Fig. 8)
14. (Caution! The strap fastener with the logo may fall in a different position on the circumference of the leg – individually for each user).

Brace selection:

1. The brace should be selected based on the dimensions shown in the size chart.
2. Make sure the device size is correct to stabilise the relevant foot appropriately. Left and right foot braces are available.
3. If the device becomes too loose or changes its position, it must be refitted starting from the first step in the section „How to put on the brace”.

Composition:

Polyethene, polyester, polyamide, chloroprene, brass, PVC, artificial leather

Cleaning procedures:

The brace must be washed by hand at low temperatures (30° C) with mild detergents, without fabric softeners. Make sure that the hook fasteners do not come into contact with the fabric; they can damage the fabric during washing. Do not use bleach or strong detergents. To dry it, lay the brace spread out on a towel and air-dry. Do not twist or wring the brace. Do not dry in a tumble dryer or on a heater. Check the condition of the brace after each wash.

CAUTION!

 The product is designed to be worn over a cotton sock. If you experience, any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the brace with warming gels or ointments. Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the member state where the user and/or patient is based.

Size table:

Brace size	Foot size
S	37-40
M	40-44
L	44-47