Intended user:

The medical device is intended for both laypersons (non-professional users) and professional users.

ATTENTION!

Do not use the device if you have circulatory disorders.

Do not use the device if it has been damaged or deformed.

Do not attempt to repair the device yourself.

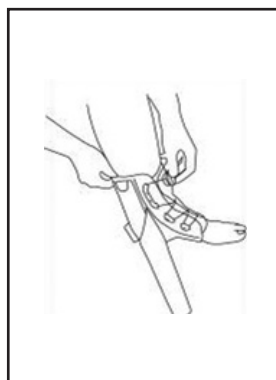
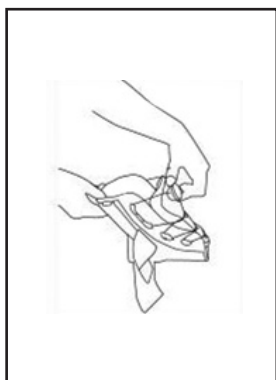
Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

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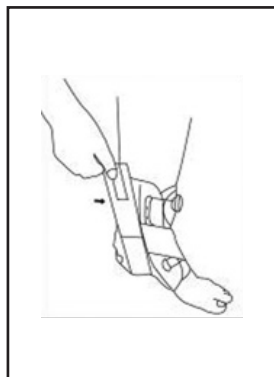
Intended use/Indications:

Used for ankle injuries: first- and second-degree sprains, ligament damage. Stabilisation in cases of permanent or temporary ankle instability. Protects the joints from excessive eversion and inversion movements. Habitual sprain of the ankle. Prevention of injury and overload of the ankle area.

Contraindications:

Unstable fractures requiring a plaster cast. The product is designed to be worn over a cotton sock. If you experience any problems related to using the product, please get in touch with your doctor. Patients with impaired blood supply to the extremities due to diabetes or peripheral vascular disease must take special care. Do not wear the orthosis on open wounds.

Device description:

- the orthosis is made of spacer fabric with excellent air and moisture permeability,
- a dial lacing system,
- stabilisation is ensured by moulded side supports,
- soft padding protects the soft tissue in the joint area,
- universal model for the right and left ankle.

Target groups of patients:

The orthosis is intended for patients requiring stabilisation of the ankle area.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Choosing your orthosis:

The orthosis should be selected based on the dimensions shown in the size chart.

How to put the orthosis on:

1. Pull the dial, loosen the laces and slide the foot in.
2. Press the dial button, then twist until you feel comfortable.
3. If you need to readjust, pull out the dial, then loosen the laces slightly and repeat the above steps.
4. Stretch the elastic strap and fasten it on the other side of the ankle.

Size chart:

Size	S	M	L
Ankle circumference (cm)	15–22	22–30	30–36

Instructions for cleaning:

Remove the supports located in the orthosis stabiliser before washing. The orthosis can be hand washed at low temperatures (30°) with a mild detergent, without fabric softeners. Do not use bleach or strong detergents. To dry it, lay the brace spread out on a towel and air-dry. Do not twist or wring the orthosis. Do not dry in a tumble dryer or on a heater. Check the condition of the orthosis after each wash.

Composition:

PE, PA, PU, PP, Cotton