

User's manual
Hip joint orthosis
Catalogue number: DRQ10B
Edition: 01.2009

The brace effectively helps rehabilitation and treatment in case of injury, diseases and dysfunction of hip joint. It controls range of motion, provides compression of soft tissues.

Indications:

Post-operative compression after transtrochanteric fractures. Post-operative compression after intertrochanteric reposition osteotomy and total hip replacement. Injuries related to excessive overload of hip joint, Hip joint degeneration. Pain syndromes of different etiology.

Contraindications:

Conditions which require immobilization or précises limitation of angular range of motion. Poor blood circulation in the limb.

Product description:

The brace consists of elastic strap which holds the hips and thigh strap made of polyamide knitted fabric and Lycra with hook and loop fastener. There is a hip pad between the straps. The pad is made of polyurethane foam laminated with velour fabrics. Additional stiff pad made of polyethylene sheeting ensures optimal hip stabilization.



Putting on:

Before the first use, you should consult a doctor or physiotherapist to adjust properly the brace (compression, stabilization - the brace may be used with or without the stiff pad). Qualified medical personnel should assist you when putting the brace for the first time. Always remember to choose the best brace size. If necessary, attach the stiff pad to the inner side of the thigh pad. Fasten the thigh strap. In order to provide optimal stabilization the straps should be well fastened but they should allow free blood circulation in the limb.

Conservation and maintenance:

Product hygiene is important for patient's rehabilitation. The brace should be washed up by hands using mild detergents (max. 35 C). Do not chlorinate. Do not use the strong chemical detergents and bleaches. Do not tumble dry. Wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat. Do not iron.



Attention!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.



Attention! The product is not fireproof. Do not use close to the open sources of fire.

Size	S	M	L	XL
Hip circumference(cm)	75,0-90,0	90,0-105,0	105,0-120,0	120,0-140,0

The final decision concerning the application and choice of the equipment belongs to a medical specialist.



mdh sp. z o.o., 90-349 Łódź, ul. Ks. W. Tymienieckiego 22/24, Tel. +4842 6748384, Fax.+48426365221,
www.mdh.pl, www.qmedinfo.pl