
PRODUKT POLSKI | POLISH PRODUCT

mdh sp z o.o.
Producent/Manufacturer:
ul. Ks. W. Tymienickiego 22/24, 90-349 Łódź,
tel. +48 42 674 83 84, fax. +48 42 636 52 21,
www.mdh.pl, www.qmedinfo.pl

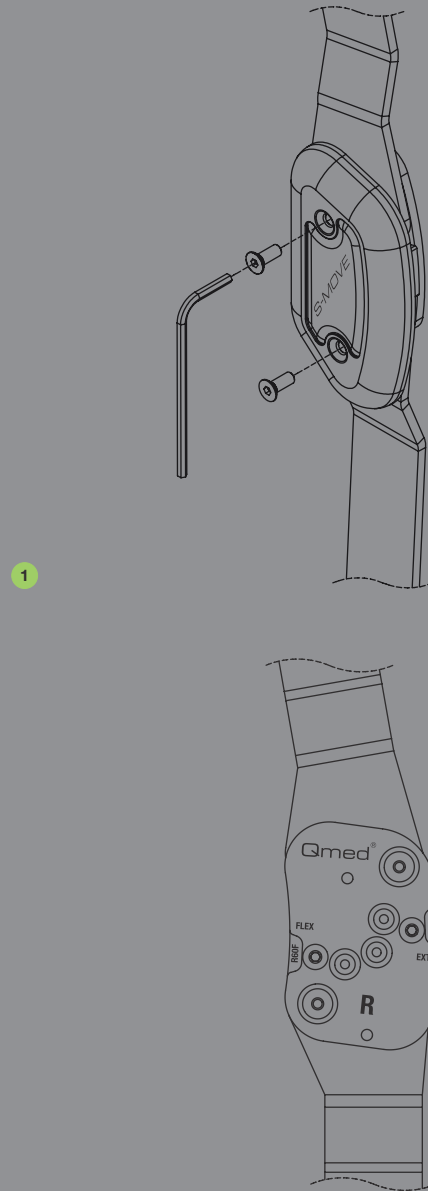1

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Instructions for use
S-Move knee joint orthosis
Stabilising knee joint orthosis (closed)
Catalogue number: DRQI0D

Application
The S-move knee orthosis can be used in a variety of therapeutic applications. It effectively supports the process of treatment and rehabilitation for injuries and dysfunctions around the knee joint, in cases not requiring immobilisation. The brace provides external stabilisation of the knee joint with two liners equipped with polycentric dials. The dials reflect the physiological movement of the joint and allow for setting two range limits (flexion and extension) within the desired therapeutic range.

Indications
Stabilisation after surgery in the final phase of functional treatment. Additional treatment for knee twists. Stabilisation in the case of degenerative changes. Acute, subacute and chronic pain syndromes of different aetiologies. Injury and dysfunction without joint immobilisation. Knee ligament system supports in overstretching and instability. Relief for ligamentous-muscular system in overloads. Patella stabilisation.

Contraindications
Conditions requiring cast immobilisation, long-healing wounds and ulcers, significant bone deformation (the orthosis cannot be adjusted to the joint).

Product description
The orthosis is made of advanced, multi-directionally flexible, breathable fabric that provides a perfect fit for the limb and the optimal level of compression while maintaining comfort. Lateral stabilisation of the joint is provided by polycentric liners. Dials allow for setting double limits to the range of extension (0°, 10°, 20°, 30°) and flexion (45°, 60°, 90°). Two additional stabilising tapes allow the adjustment of compression. The brace has a shaped patella opening, which provided lateral stabilisation of the patella and relief of the femoropatellar joint.

Application manual
Open the stabilising straps. Pull the orthosis on, so the patella is in the opening, and the side liners are symmetrically placed on both sides of the knee joint. Fasten the stabilising straps, adjusting the force to your needs.

Adjusting the range of motion
The changes and limiting of angular movement is prescribed by a doctor. The brace is provided with the limit of extension set to 0°. If your doctor tells you to limit the angular range of motion: Loosen two screws on the dial. Loosen the top screw if you want to change the extension range. Remove the wedge held by the screw. Replace it with a wedge with appropriate agular range marking. Tighten the screw. If you want to change the flexion range, loosen the screw in the back part of the dial. Remove the wedge held by the screw. Replace it with a wedge with appropriate agular range marking. Tighten the screw. When the range of flexion and extension angle is set within the therapeutic range, tighten the screws holding the dial's cover. Note that the angular range should be determined before the orthosis is applied, and that both dials have to be set to the same value.

Size					* The size is selected based on the circumference of the thigh (in cm), measured 15 cm above the centre of the patella.
1	2	3	4	5	
36-39	40-43	44-48	49-54	55-60	

Maintenance
Remove the liners before washing. Hand wash in lukewarm water with mild detergent. Do not use chlorine. Do not wring, gently squeeze. Dry in the extended position away from any artificial heat sources. Do not iron.

⚠ NOTE. Failure to observe good personal hygiene or improper use of detergents may cause abrasions or other ailments associated with increased sweating and bacterial flora. Do not use the orthosis on warming gels and ointments. Keep in mind that detergent residue may irritate the skin and damage the material of the orthosis.

FINAL DECISION CONCERNING THE USE AND SELECTION OF REHABILITATION EQUIPMENT IS MADE BY A SPECIALIST OR PHYSICAL THERAPIST.

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