DE

Wydanie: 17.11.2021

Edition: 17.11.2021

Ausgabe: 17.11.2021

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Producent / Manufacturer/ Hersteller:

MDH Sp. z o.o.
ul. Maratońska 104
94-007 Łódź
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.qmedbymejra.pl

Anzeigen

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EN

Indications

Treatment of wrist and thumb injuries and distal forearm, chronic wrist and thumb instability with pain syndrome, convalescence after surgical or conservative treatment, wrist and thumb protection in sports.

Indications (long)

Treatment of wrist and thumb injuries and distal forearm, chronic wrist and thumb instability with pain syndrome, convalescence after surgical or conservative treatment, wrist and thumb protection in sports.

⚠ CAUTION!

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

⚠ CAUTION!

Not observing the rules of personal hygiene and cleaning the orthosis with inappropriate detergents may result in skin abrasion or other effects related to sweating. Please, do not use the warming ointment.

Application instructions

1. Set the straps to most loosening setting for easier application.
2. Match the universal thumb-up splint.
3. Slide hand into brace, gently guiding the thumb into the thumb support/thumb hole.
4. Adjust the brace to a snug fit.

Maintenance

Orthosis can be washed by hand in a very mild detergents at temperatures up to 35°C, after removing the aluminum stay. Do not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat. Dry in exploded form. Do not dry.

Size

Universal.