#### PRODUCT DESCRIPTION:

Spincare Flex III is made of elastic, flexible straps. The fabric carries humidity and warmth away. Four semi-elastic stays provide passive stabilization. Two tightening straps enable optimal adjustment and compression.

#### PUTTING ON:

Always make sure that the size of brace has been carefully chosen which is essential for proper functioning of the brace. In order to put on the brace, the patient should lie down. Put the brace so that it would be placed 2 cm above the coccygeal vertebrae - brace's label (you can find it inside the brace) should be directed upward. Back stays should adhere to lumbosacral area. Fasten the belt and then additional straps which regulate the dorsal compression. Make sure that the compression provides optimal stabilization (partly reduces the movement of L-S spine) but does not disturb breathing. Time of using the brace should be precised by medical specialist and should be accompanied by rehabilitation exercises program.

Size	S	M	L	XL	XXL
Waist circumference (cm)	56 – 71	69 - 84	81 - 96	94 - 109	107 - 122

#### CONSERVATION AND MAINTENANCE:

The brace should be washed using gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat sources. Dry in horizontal position. Do not iron.

#### ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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## INSTRUCTIONS FOR USE

### SPINCARE FLEX III - Lumbosacral orthosis

SPINCARE FLEX III effectively helps rehabilitation and treatment in case of injuries, diseases and dysfunctions of intervertebral joints, intervertebral disc, muscle attachments, ligaments of lumbar area and ligaments of lumbosacral passage. The dynamic stabilization enables effective control of movement range, relief and posture correction which alleviate symptoms connected to degenerative changes and functional disorders of spinal segment.

#### INDICATIONS:

- Non-invasive treatment of dysfunctions of the lumbosacral segments of various etiology and intensity.
- Stabilisation of lumbar spine – instability after surgeries, treatment after neurological and orthopaedic surgeries.
- Functional treatment, protection against further injuries, overstrain.
- Inborn defects of lumbar segment.
- Chronic pain.

#### CONTRAINDICATIONS:

- Conditions which require immobilisation of L-S section.