PRODUCT DESCRIPTION:

Spinactive is made of flexible straps. The fabric carries humidity and warmth away. Additional, removable straps enable easy adjustment of dorsal compression. Four semi-elastic stays provide passive stabilization. The brace helps to balance the compression on Intervertebral disk and intervertebral joints. It supports and relieves weakened and tense muscles.

PUTTING ON:

Do not forget to make sure that the brace size has been chosen carefully that is essential for proper functioning and comfort of use. In order to put on the brace, the patient should lie down. Put the brace so that it would be placed 2 cm above the coccygeal vertebrae. The brace's label (you can find it inside the brace) it should be directed upward. Back stays should adhere to lumbosacral area. Fasten the belt and then additional straps which regulate dorsal compression. Make sure that the compression provides optimal stabilization (partly reduces the movement of L-S spine) but does not disturb breathing. Time of using the brace should be precised by medical specialist and should be accompanied by proper rehabilitation exercises program.

Size	1	2	3	4	5	6
Waist circumference (cm)	52 - 62	63 - 74	75 - 89	90 - 105	106 - 120	121 - 140

MAINTENANCE:

The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat sources. Dry in horizontal position. Do not iron.

ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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INSTRUCTIONS FOR USE

SPINACTIVE – lumbosacral orthosis

It helps effectively with rehabilitation and treatment in case of injuries, diseases and dysfunctions of intervertebral joints, intervertebral disc, muscle attachments, ligaments of lumbar area and ligaments of lumbosacral passage. The dynamic stabilization enables effective control of movement range, relief and posture correction which alleviate symptoms connected to degenerative changes and functional disorders of spinal segment.

INDICATIONS:

- Intense lumbar pain, disc disease, sciatica, painful degeneration of spine, straining syndromes.
- Non-operative treatment of spinal segment dysfunctions of different etiology.
- Stabilization of lumbar area in case of segmental instability after surgeries.
- Follow-up treatment after neurosurgical/orthopaedic interventions.
- Prophylaxis against injuries, overload.
- Congenital defects of lumbar area (with or without pain syndromes).
- Degenerative spinal disorders with pain syndrome.
- Disorders of abdominal muscle tension (surgical intervention, risk of ruptures).