FUNCTIONING:

- Reducing the amplitude related to bending-straightening of lumbar spine.
- Equalizing the pressure on intervertebral disk and intervertebral joints.
- Supporting and relieving the weakened and tense muscles.
- Mechanical stabilisation.

USE:

Do not forget to make sure that the brace size has been chosen carefully that is essential for proper functioning of the brace. In order to put on the brace, the patient should lie down. Put the brace so that it would be placed 2 cm above the coccygeal vertebrae - the brace's label (you can find it inside the brace) it should be directed upward. Back liners should adhere to lumbar-sacral area. Fasten the belt and the additional belts which regulate dorsal pressure. Make sure that the pressure provides optimal stabilization (partly reduces the movement of L-S spine) but does not disturb breathing. Time of using the brace should be precised by medical specialist and should be accompanied by rehabilitation exercises program.

Size	1	2	3	4	5	6
Waist circumference (cm)	52 - 62	63 - 74	75 - 89	90 - 105	106 - 120	121 - 140
Height (cm)	25					

MAINTENANCE:

The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat sources. Dry in horizontal position. Do not iron.

ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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INSTRUCTIONS FOR USE

SPINACTIVE – LS II lumbosacral orthosis

Dynamic control of movement range. Optimal stabilization. Breathable fabric It helps effectively with rehabilitation and treatment in case of injuries, diseases and dysfunctions of intervertebral joints, intervertebral disc, muscle attachments, ligaments of lumbar area and ligaments of lumbosacral passage. The dynamic stabilization enables effective control of movement range, relief and posture correction which alleviate symptoms connected to degenerative changes and functional disorders of spinal segment.

INDICATIONS:

- Nonoperative treatment of spinal segment dysfunctions of different etiology.
- Intense lumbar pain, disc disease, sciatica, straining syndromes.
- Stabilization of lumbar area in case of segmental instability after surgeries.
- Follow-up treatment after neurosurgical/orthopedic interventions.
- Prophylaxis against injuries, overload.
- Congenital defects of lumbar area (with or without pain syndromes).
- Degenerative spinal disorders with pain syndrome.
- Disorders of abdominal muscle tension (surgical intervention, risk of ruptures).

PRODUCT DESCRIPTION:

The Spinactive LS II is made of flexible bands. The fabric carries humidity and warmth away. Two additional straps allow easy adjustment of dorsal pressure. Four semi-elastic liners provide passive stabilization.