#### PUTTING ON:

- Do not forget to make sure that the brace size has been chosen carefully.
- In order to put on the brace, the patient should lie down.
- Put the brace so that it would be placed 2 cm above the coccygeal vertebrae - the brace's label (you can find it inside the brace) should be directed upward. The lateral stays should adhere to T-L-S area.
- Fasten the belts and regulate the dorsal compression.
- Make sure that the brace provides optimal stabilization but does not disturb breathing.
- The time of using the brace should be precised by medical specialist and should be accompanied by rehabilitation exercises program.

Size	1	2	3	4	5	6
Waist circumference (cm)	52 - 62	63 - 74	75 - 89	90 - 105	106 - 120	121 - 140
Height (cm)	31					

#### CONSERVATION AND MAINTENANCE:

The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat sources. Dry in horizontal position. Do not iron.

#### ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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## INSTRUCTIONS FOR USE

### SPINCARE FLEX – thoracic-lumbosacral orthosis

Spincare Flex brace effectively helps with rehabilitation and treatment in case of injuries, diseases and dysfunctions of intervertebral joints, intervertebral disc, muscle attachments, ligaments of lumbar area and ligaments of lumbosacral passage. The dynamic stabilization enables effective control of movement range, relief and posture correction which alleviate symptoms connected to degenerative changes and functional disorders of spinal segment.

#### INDICATIONS:

Intense spinal pain, disc disease, painful degeneration of spine, stabilization after surgery during functional treatment, spinal stenosis.

#### PRODUCT DESCRIPTION:

The Spincare Flex is made of flexible straps. The fabric carries humidity and warmth away. Five semi-elastic asymmetric stays provide passive stabilization and allow to adjust height and stabilization strength to chosen TLS section. The brace helps to balance the compression on intervertebral disk and intervertebral joints. It supports and relieves weakened and tense muscles.