#### FUNCTIONING:

- Reducing the amplitude related to bending-straightening of T-L-S spine.
- Equalizing the pressure on intervertebral disk and intervertebral joints.
- Supporting and relieving the weakened and tense muscles.
- Helping in stabilization of T-L-S spine.

#### USE:

Do not forget to make sure that the brace size has been chosen carefully, that is essential for proper functioning of the brace. In order to put on the brace, the patient should lie down. Put the brace so that it would be placed 2 cm above the coccygeal vertebrae - brace's label (you can find it inside the brace) should be directed upward. Back liners should adhere to TLS area. Fasten the belt. Fasten the belts of upright keeper, place them in front of shoulder joint, cross them at the back and fasten them at the chest. Then, fasten the belt which regulates pressure and adjust it. Make sure that the belt provides optimal stabilization (partly reduces the movement of L-S spine) but does not disturb breathing. The time of using the brace should be precised by medical specialist and should be accompanied by rehabilitation exercises program.

Size	S	M	L	XL	XXL
Waist circumference (cm)	80 - 90	90 - 100	100 - 110	110 - 120	120 - 130

Height of lumbosacral brace: 30 cm / Height of lumbosacral brace with upright keeper: 54 cm

#### CONSERVATION AND MAINTENANCE:

Move out the liners before washing. The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial warmth source. Dry in horizontal position. Do not iron.

#### ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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## INSTRUCTIONS FOR USE

### SPINCARE HIGH – thoracic-lumbosacral orthosis

Dynamic control of movement range. Optimal stabilization. Breathable fabric. Spincare High TLSO support (Taylor type) has a wide therapeutic use. As semi-rigid support it helps effectively with rehabilitation and treatment in case of injuries dysfunctions of T-L-S area. It provides optimal stabilization and by controlling the range of bending, straightening and trunk lateral movements, it determinates "safe movement range". The construction enables adjusting the height and stabilization strength to the individual therapeutic needs which means that the movement of T-L-S spine is adjusted to the individual therapeutic range.

#### INDICATIONS:

Intense and chronic pain syndroms of lumbar area. Discopathy. Degenerative spinal disorders with pain syndrome. Stenosis of vertebral canal. Stabilization after surgery and during functional treatment. Thoracic kyphosis. Scheuermann's disease. Osteoporosis. Postural defects. Weakness of abdomen muscles.

#### PRODUCT DESCRIPTION:

Spincare High is made of flexible, tensile bands. The fabric carries humidity and warmth away. Four semi-rigid and two rigid liners provide optimal stabilization. Spincare High consists of two belts, suspenders fastened in front and crossed at the back. Additional bands allow to regulate direction of pressure strength.