#### INDICATIONS:

Intense and chronic lumbar pains, discopathy, disc diseases, spinal stenosis, stabilization after surgery and during functional treatment. Thoracic kyphosis, Scheuermann's disease, osteoporosis, postural defects, weakness of abdomen muscles, ankylosing spondylitis.

#### PUTTING ON:

Do not forget to make sure that belt size has been chosen carefully, that is essential for proper functioning of the brace. If you want to use only the LS belt, please, remember to put two rigid straps in the middle tunnels of the brace. Put the LS belt so that its bottom edge would be 2 cm above caudal bone (width of two fingers above gluteal fissure)- brace's label (you can find it inside the brace) should be directed upward. Fasten the LS straps. Make sure that the belt provides optimal stabilization (partly reduces the movement of L-S spine) but does not disturb breathing. In order to use the upright keeper, remove the metal stays from the LS belt. Ask a qualified personnel to fasten the upright keeper – stays should be located centrally along the spine (they should adhere to the TLS area). In order to provide optimal stabilization and correction the upper edge of upright keeper stays should be at the height of upper angle of shoulder-blade. Fasten the straps, then the suspenders – cross the suspenders at the back, move them on the chest and fasten. The time of using the brace should be precised by medical specialist and should be accompanied by rehabilitation exercises program.

Size	S	M	L	XL	XXL
Waist circumference (cm)	80 - 90	90 - 100	100 – 110	110 - 120	120 - 130

Height of lumbosacral brace: 31 cm / Height of lumbosacral brace with upright keeper: 54 cm

#### CONSERVATION AND MAINTENANCE:

Always remove the stays before washing. The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat sources. Dry in horizontal position. Do not iron.

#### ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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## INSTRUCTIONS FOR USE

### SPINCARE HIGH PLUS – thoracic-lumbosacral orthosis

Spincare High Plus is TLS support of wide therapeutic use. As semi-rigid support it helps effectively with rehabilitation and treatment in case of injuries/ dysfunctions of T-L-S area. It provides optimal stabilization and by controlling the range of bending, straightening and trunk lateral movements, it determinates "safe movement range". The construction enables adjusting the height and stabilization strength to the individual therapeutic needs which means that the movement of T-L-S spine is adjusted to the individual therapeutic range.