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EN

### Intended use/Indications:

Post-operative treatment of fractures of the head of the humerus, glenoid cavity, shoulder dislocations. Degenerative shoulder changes and associated pain. Habitual shoulder dislocations Recovery after glenohumeral capsular reconstruction. Protection against shoulder dislocation in upper limb paralysis of various aetiologies.

### Contraindications:

Conditions requiring immobilisation of the shoulder. Wounds and skin lesions within the area of the orthosis.

### Device description:

- the orthosis consists of an air-permeable stabiliser,
- the straps ensure stabilisation of the shoulder and the acromioclavicular joint and allow for individual adjustment of the orthosis,
- universal size.

### Target groups of patients:

The orthosis is intended for patients requiring stabilisation of the shoulder area.

### Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

### How to put the orthosis on:

1. Put the cuff on the arm and shoulder.
2. Fasten the shoulder strap.
3. Adjust the arm to the correct position.
4. Then fasten strap 2 on the chest.

### Size chart:

Universal size.

### Instructions for cleaning:

Remove the supports located in the orthosis stabiliser before washing. The orthosis can be hand washed at low temperatures (30°) with a mild detergent, without fabric softeners. Do not use bleach or strong detergents. To dry it, lay the brace spread out on a towel and air-dry. Do not twist or wring the orthosis. Do not dry in a tumble dryer or on a heater. Check the condition of the orthosis after each wash.

### Composition:

PA, SBR, PE

### Intended user:

The medical device is intended for both laypersons (non-professional users) and professional users.

### ATTENTION!

Do not use the device if you have circulatory disorders. Do not use the device if it has been damaged or deformed. Do not attempt to repair the device yourself. Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE