Intended user:

The medical device is intended for both laypersons (non-professional users) and professional users.

ATTENTION!

Do not use the device if you have circulatory disorders.

Do not use the device if it has been damaged or deformed.

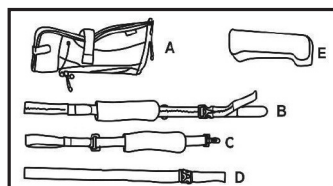
Do not attempt to repair the device yourself.

Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.


IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

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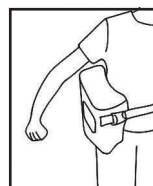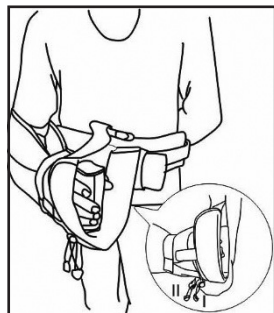
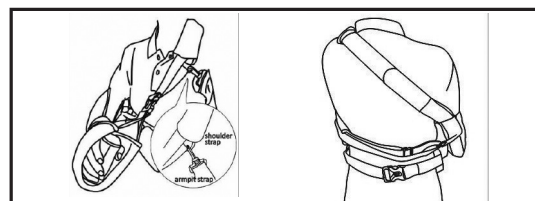
EN

SHOULDER IMMOBILIZER 15 SHOULDER ABDUCTION ORTHOSIS

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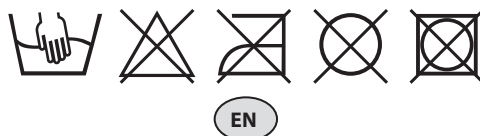
EN

Intended use:

A shoulder abduction orthosis with wide therapeutic use. Effectively supports treatment and rehabilitation of injuries and dysfunctions of the shoulder, for conditions requiring immobilisation in abduction.

Indications:

- stabilisation of the upper limb in abduction at the shoulder joint after

trauma (sprains, dislocations) or surgery.

- Recovery after rotator cuff repair.
- Immobilisation and relief of the upper limb for inflammatory lesions of the shoulder.

Contraindications:

Paresthesias and lymphostasis. Skin diseases and injuries in the area of the orthosis. Conditions requiring immobilisation of the upper limb in another therapeutic position.

Device description:

- the orthosis is made of soft velour,
- immobilisation of the shoulder at a 15° abduction,
- a comfortable foam padding provides more support.

Target groups of patients:

The orthosis is intended for patients requiring stabilisation of the shoulder area.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Size chart:

Universal size.

How to put the orthosis on:

1. The orthosis consists of (A) sling, (B) shoulder strap, (C) underarm strap, (D) pelvic strap, (E) padding.
2. Fasten the pelvic strap on both sides of the padding. Adjust its length and fix it at waist height.
3. Slip your hand into the sling and secure it to the padding. Roll out the cuff and fasten it with Velcro. Adjust the width of the cuff (I) and fix the grip pad (II) by pulling the elastic straps under the cuff.
4. Attach the shoulder strap to the opposite shoulder by fastening it to the upper sling attachment.
5. Attach the shoulder strap to the opposite shoulder by fastening it at the shoulder strap fastening ring, passing the strap under the armpit to the rear fastening ring on the sling. Fasten the strap securely in this position.

Instructions for cleaning:

Padding and tightening straps should be hand-washed at low temperatures (30°) with mild detergents, without softeners. Before washing, secure, fasten all Velcro fasteners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry. Keep away from artificial sources of heat. Check the condition of the orthosis after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

Composition:

PU, PA, POM, PE, PP, SBR, St, cotton