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EN


Product description:

Lightweight open structure of the brace is made of foam, nylon, polyester. It is equipped with two underwire and anatomically profiled stay. Orthosis ensures free movement of the thumb. Construction of the splint guarantees immobilization of the wrist joint in a functional position. The brace is an universal supply for the right and left hand. Three bands allow to adjust the orthosis to the patient's hand. Soft lining ensures comfort of use.

Indications:

The hand and forearm orthosis is used in case of soft-tissue irritations of the wrist, for the conservative immobilization after distorsions and in case of tendovaginitis, carpal tunnel syndrome and rheumatoid arthritis.

Putting on:

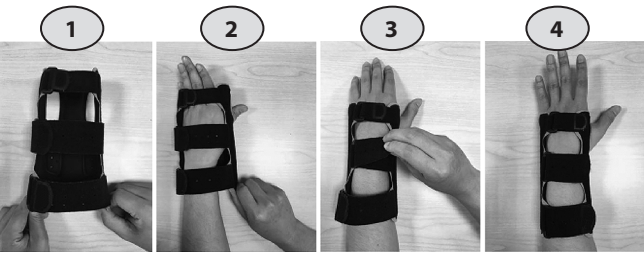1. Set the straps to most loosening setting for easier application.
2. Slide affected wrist into the brace.
3. Adjust the brace to a snug fit.
4. Properly fitted orthosis.

Maintenance:

Orthosis can be washed by hand in a very mild detergents at temperatures up to 35°C, not chlorinate, do not dry clean, do not wring, do not spin, squeeze gently before drying. Dry away from artificial sources of heat. Dry in exploded form. Do not dry.

ATTENTION!

Not observing the rules of personal hygiene and cleaning the orthosis with inappropriate detergents may result in skin abrasion or other effects related to sweating. Please, do not use the warming ointment.

FINAL DECISION CONCERNING THE USE AND SELECTION OF REHABILITATION EQUIPMENT IS MAD BY A SPECIALIST OR PHYSICAL THERAPIST

Size chart:

Size	Wrist circumference (cm)
S	13-15 cm
M	15-18 cm
L	18-20 cm
XL	20-22 cm