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EN

Indications:

The UNO brace is intended to stabilise individual fingers of the hand. It is intended for use in deformities and injuries of the finger joints, chronic joint instability, traumatic injuries to tendons, recovery of injuries after removal of plaster cast, recovery of injuries after surgery.

Contraindications:

- Skin lesions, edema, varicose veins, blood clotting disorders.
- Allergic reaction to materials used.

Intended user:

The medical device is intended for the general public (a non-professional user).

Target patient groups:

It is intended for patients with deformed and damaged finger joints, chronic joint instability, traumatic injuries to tendons, for the recovery of injuries after removal of plaster cast, for the recovery of injuries after surgery.

Disposal of the medical device:

Disposal of the device must be carried out in line with the applicable local environmental regulations.

How to put on the brace:

The instructions below show how to put the brace on the right hand with the index finger immobilised. (Table 1)

1. Place the hand on the body of the brace so that the channels for placing the liners are on the top side of the hand (the inlet of the channel facing the fingers) and fasten the first, top Velcro around the fingers. Fig. 1
2. Fasten the lower Velcro around the wrist. Fig. 2
3. Insert the liner into the channel corresponding to the position of the finger to be immobilised. Fig. 3
4. Wrap around the chosen finger and fasten the Velcro fasteners. Fig. 4

Brace selection:

1. The brace should be selected based on the dimensions shown in the size chart.
2. If the device becomes too loose or changes its position, it must be refitted starting from the first step in the section, How to put on the brace’.
3. The UNO brace set includes one splint to stabilise one selected finger. If more fingers need to be stabilised, individual rails can be purchased - as an accessory. A maximum of 4 splints can be fitted to the Orthosis (for all fingers except the thumb, fitting as shown in Table 1).

Size chart:

Universal size, suitable for right and left hand.

Size	Circumference of metacarpus
1	18-21 cm
2	22-25 cm

Cleaning procedures:

The brace must be washed by hand at low temperatures (30° C) with mild detergents, without fabric softeners. Do not use bleach or strong detergents. To dry it,

lay the brace spread out on a towel and air-dry. Do not twist or wring the brace. Do not dry in a tumble dryer or on a heater. Check the condition of the brace after each wash. The metal rails must be removed from the brace before washing.

Composition:

PES, PA, CR, AL.

ATTENTION!

In case of a skin allergy, stop using the device immediately and consult a doctor. Do not use for more than 24 hours continuously. Do not use the device on children without adult supervision. Do not use the device if it has been damaged or distorted. Do not attempt to repair the device yourself. Do not use the device on patients with superficial and deep sensation disorders without guardian supervision. Do not use the device in the wrong size or in a way for which it is not intended.

ATTENTION!

The product is designed to be worn directly on the hand. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the brace with warming gels or ointments. Any serious incident that occurs in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

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