

PRODUKT POLSKI | POLISH PRODUCT

mdh sp z o.o.
Producent/Manufacturer:
ul. Ks. W. Tymienieckiego 22/24, 90-349 Łódź,
tel. +48 42 674 83 84, fax. +48 42 636 52 21,
www.mdh.pl, www.qmedinfo.pl

1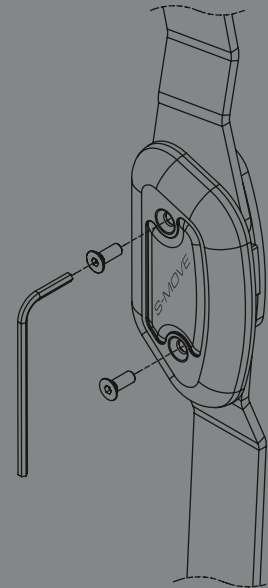

3

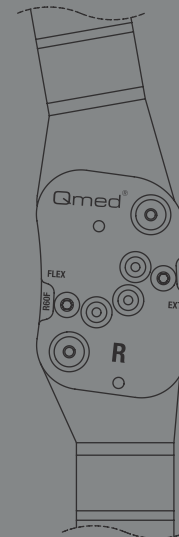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Instructions for use
S-Move knee joint orthosis
Stabilising knee joint orthosis (open)
Catalogue number: DRQI0I

Application
The S-move knee orthosis can be used in a variety of therapeutic applications. It effectively supports the process of treatment and rehabilitation for injuries and dysfunctions around the knee joint, in cases not requiring immobilisation. The brace provides external stabilisation of the knee joint with two liners equipped with polycentric dials. The dials reflect the physiological movement of the joint and allow for setting two range limits (flexion and extension) within the desired therapeutic range.

Indications
Stabilisation after surgery in the final phase of functional treatment. Additional treatment for knee twists. Stabilisation in the case of degenerative changes. Acute, subacute and chronic pain syndromes of different aetiologies. Injury and dysfunction without joint immobilisation. Knee ligament system supports in overstreessing and instability. Relief for ligamentous-muscular system in overloads. Patella stabilisation.

Contraindications
Conditions requiring cast immobilisation, long-healing wounds and ulcers, significant bone deformation (the orthosis cannot be adjusted to the joint).

Product description
As it is made of advanced, breathable AirX knitted multidirectional elastic fabric, the orthosis ensures suitable comfort of use. Additional possibility of opening the structure facilitates its putting on and fitting, and two strip stays ensure appropriate level of compression on the limb and adjustment of the stabilization strength. Lateral stabilizatiion of the joint is ensured by polycentric liners. Controllers allow double limitation of the range of motion: extension: 0°, 10°, 20°, 30° and flexion: 45°, 60°, 90°. The orthosis has a contoured orifice for the patella, ensuring its lateral stabilization and relief for the patellofemoral joint.

Application manual
Open the orthosis. Screw off the controller covers and make sure the settings are the same on both of them. Put it on the lower limb, so as to place the patella in the contoured orifice. Fasten the grip straps. Tighten the straps adjusting the stabilizing force to individual needs.

Adjusting the range of motion
The changes and limiting of angular movement is prescribed by a doctor. The brace is provided with the limit of extension set to 0°. If your doctor tells you to limit the angular range of motion: Loosen two screws on the dial. Loosen the top screw if you want to change the extension range. Remove the wedge held by the screw. Replace it with a wedge with appropriate agular range marking. Tighten the screw. If you want to change the flexion range, loosen the screw in the back part of the dial. Remove the wedge held by the screw. Replace it with a wedge with appropriate agular range marking. Tighten the screw. When the range of flexion and extension angle is set within the therapeutic range, tighten the screws holding the dial's cover. Note that the angular range should be determined before the orthosis is applied, and that both dials have to be set to the same value.

Size					* The size is selected based on the circumference of the thigh (in cm), measured 15 cm above the centre of the patella.
1	2	3	4	5	
36-39	40-43	44-48	49-54	55-60	

Maintenance
Remove the liners before washing. Hand wash in lukewarm water with mild detergent. Do not use chlorine. Do not wring, gently squeeze. Dry in the extended position away from any artificial heat sources. Do not iron.

⚠ NOTE. Failure to observe good personal hygiene or improper use of detergents may cause abrasions or other ailments associated with increased sweating and bacterial flora. Do not use the orthosis on warming gels and ointments. Keep in mind that detergent residue may irritate the skin and damage the material of the orthosis.

FINAL DECISION CONCERNING THE USE AND SELECTION OF REHABILITATION EQUIPMENT IS MADE BY A SPECIALIST OR PHYSICAL THERAPIST.

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