UWAGA

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ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist.
When used for the first time, the support should be put on by the experienced health-care personnel.
Before the first use, please, read the user's manual carefully.
If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

Knee support with open patella has wide therapeutic use. It helps rehabilitation and treatment of knee injuries and dysfunctions which do not require immobilization. It ensures external stabilization. Polycentric liners enable unhindered bending/straightening movement. Special construction allows to adjust the stabilization strength to individual needs.

Indications:

Intense, chronic pain syndromes of relatively low intensity and of various etiology.
Postoperative stabilization during final phase of functional treatment.
Injuries and dysfunctions which do not require joint immobilization.
Support for the ligament apparatus of knee joint.

Contraindications:

Conditions which require immobilization of knee joint
Difficult to heal wounds, sores or other skin changes

Product description:

Knee support is made of polyamide knitted fabric laminated with a layer of perforated neoprene. It maintains natural body warmth and even raises it locally. Polycentric liners provide lateral stabilization. The brace is equipped with patellar opening and opening for patellar fossa which enable unhindered knee flexion.

Use:

Put on the brace so that the knee would be placed in patella opening.
Fasten the Velcro bands adjusting the pressure strength to individual needs.

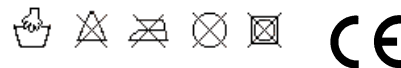
Maintenance:

Remove the liners before washing. The brace should be washed up by hand using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying.
Dry in horizontal position. Dry in a distance of artificial sources of heat.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

	S	M	L	XL
Obwód uda (cm)	41 - 44	44 - 47	47 - 50	50 - 53

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