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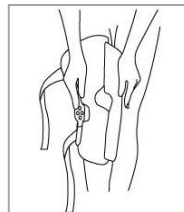
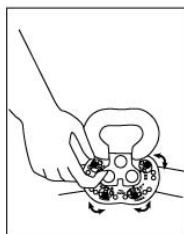
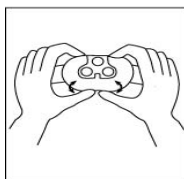
### ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel.

Before the first use, please, read the user's manual carefully.

If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

Knee brace with patellar opening has wide therapeutic use. As semi-rigid brace it effectively helps rehabilitation and treatment of knee injuries/dysfunctions in conditions which require limitation of angular range of motion. Polycentric liners enable external stabilization and control of flexion/extension movement. Construction of the brace allow to adjust compression strength to individual needs.

### Indications:

Intense, chronic pain syndromes of relatively low intensity and of various etiology. Functional treatment of knee joint injuries which require control of angular range of motion.

Injuries and dysfunctions which do not require joint immobilization.

Support for the ligament apparatus of knee joint.

### Contraindications:

Conditions which require immobilization of knee joint.

Difficult to heal wounds, sores or other skin changes.

### Product description:

Brace is made of polyamide knitted fabric laminated with neoprene. It maintains natural body warmth and ensures optimal compression of soft tissue. The lines with polycentric hinges provide lateral stabilization and allow to control flexion/extension movement. Patellar opening and bands enable optimal adjustment of the brace and regulation of stabilization strength. The brace is equipped with patellar opening and opening for patellar fossa which enable unhampered knee flexion.

### Use:

Unfasten the brace.

Remove the cover from the hinge.

Choose the required angular range of motion.

Put on the brace so that the patella would be placed in the patellar opening.

Fasten the Velcro bands.

Fasten the bands adjusting the stabilization strength to individual needs.

### Maintenance:

Remove the liners before washing. The brace should be washed up by hand using mild detergents. Do not chlorinate. Do not whirlpool, wring gently before drying. Dry in horizontal position. Dry in a distance of artificial sources of heat.

### ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

	S	M	L	XL
Obwód uda (cm)	41 - 44	44 - 47	47 - 50	50 - 53

In order to choose a proper size, measure the thigh circumference 15 cm above the center of

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THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

Producent/Manufacturer: mdh sp. z o.o., Ul. Maratońska 104, 94-007 Łódź, tel. +4842 6748384, www.mdh.pl, www.qmedbymejra.pl.

mdh Ltd. Ul. Maratonska 104, 94-007 Lodz, Poland, phone: +4842 212 38 04, fax. +4842 6740499

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