5. Tighten the screw. Repeat on the back of the dial, placing a wedge with the mark of the chosen flexion angle range.
6. Unfasten all stabilising straps.
7. Fit the orthosis so that the frame components rest on the front of the thigh and on the calf, and the side splints are located symmetrically on both sides of the knee. The axis of the dial should be at the level of the upper edge of the patella.
8. Fasten the stabilising straps in the following order: front femoral strap, back tibial strap, front upper tibial strap, back lower femoral strap, back upper femoral strap, front lower tibial strap.

The orthosis is equipped with additional padding set, which allows adjusting its circumference around the knee.

Instructions for cleaning:

Detach the padding from the stabilising splint before washing. Padding and tightening straps should be hand-washed at low temperatures (30°) with mild detergents, without softeners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry. Keep away from artificial sources of heat. Check the condition of the orthosis after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

Composition:

Al, PU, ST, PP, PE, PA


Intended user:

The medical device is intended for both laypersons (non-professional users) and professional users.

ATTENTION!

Do not use the device if you have circulatory disorders.
Do not use the device if it has been damaged or deformed.
Do not attempt to repair the device yourself.
Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

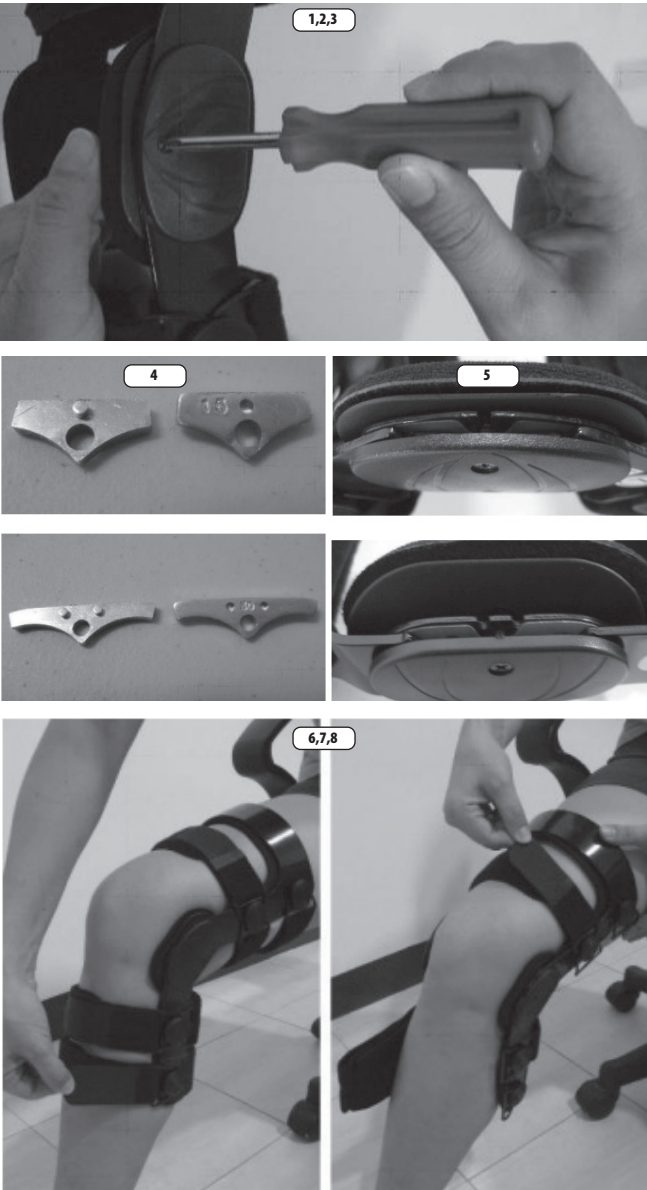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**Instructions for use:**

The SECOND orthosis effectively supports treatment and rehabilitation of the knee in cases of injury, requiring a restricted angular range of motion.

Indications:

- stabilisation after ACL, PCL, MCL, LCL injuries,
- protection of the knee during post-traumatic return to sporting activity,
- recovery after knee ligament reconstruction surgery.

Contraindications:

Conditions requiring complete immobilisation of the knee, wounds that are difficult to heal or skin lesions in the area of the orthosis.

Device description:

- an anatomical design, lightweight aluminium frame,
- a polycentric locking system enabling dual restriction of the angular range of motion in flexion (0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°) and extension (0°, 10°, 20°, 30°, 40°) and immobilisation at 0°, 10°, 20°, 30°,
- stabilisation straps to ensure the correct positioning and effectiveness of the orthosis,
- air-permeable condyle pads (under the dial) to ensure comfort – preventing skin irritation and chafing,
- removable padding included,
- the orthosis is available for the left and right limb.

Target groups of patients:

The orthosis is intended for patients with dysfunction of the knee, after injuries, diseases, to support treatment and rehabilitation by providing limitation of flexion and extension of the limb and lateral stabilisation of the knee.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Choosing your orthosis:

1. The first time the orthosis is fitted, it should be done upon prescription and under the supervision of qualified medical personnel.
2. The orthosis should be selected based on the dimensions shown in the size chart.
3. If the device becomes too loose or changes its position, it must be refitted starting from the first step in the section, How to put the orthosis on.

Size chart:

The orthosis is available for the left and right limb.

Size	XS	S	M	L
Thigh circumference [cm]	33–39	39–48	48–57	57–67

Thigh circumference measurement – 15 cm above the centre of the kneecap.

How to put the orthosis on:

1. Before fitting the orthosis, set the correct angular motion restrictions as recommended by your doctor or physiotherapist.
2. Make sure the settings on both dials are symmetrical.
3. To set the correct angular restriction, unscrew the screws located on the dial cover.
4. Place a wedge on the front of the dial to mark the selected angular range of the extension.