How to put the orthosis on:

1. Unfasten all straps.
2. Sit down and bend your leg at a 30° angle.
3. Put the orthosis on the knee.
4. Align the axis of the hinge with the axis of rotation of the knee.
5. Fasten strap 1 at the back of the calf. Then fasten strap 2 above the knee.
6. Make sure the front of the orthosis is pressed flat against the thigh and fasten strap 3.
7. Then fasten the front calf strap 4 at the front below the knee.
8. Fasten the back calf strap 5.
9. Remember to fasten all straps in the correct order.
10. While standing, fasten strap 6 at the front above the knee.

Instructions for cleaning:

Detach the padding from the stabilising splint before washing. Padding and tightening straps should be hand-washed at low temperatures (30°) with mild detergents, without softeners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry. Keep away from artificial sources of heat. Check the condition of the orthosis after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

Composition:

PE, EVA, PA, PU, POM, AL, ABS, cotton

Intended user:


The medical device is intended for both laypersons (non-professional users) and professional users.

ATTENTION!

Do not use the device if you have circulatory disorders.
Do not use the device if it has been damaged or deformed.
Do not attempt to repair the device yourself.
Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

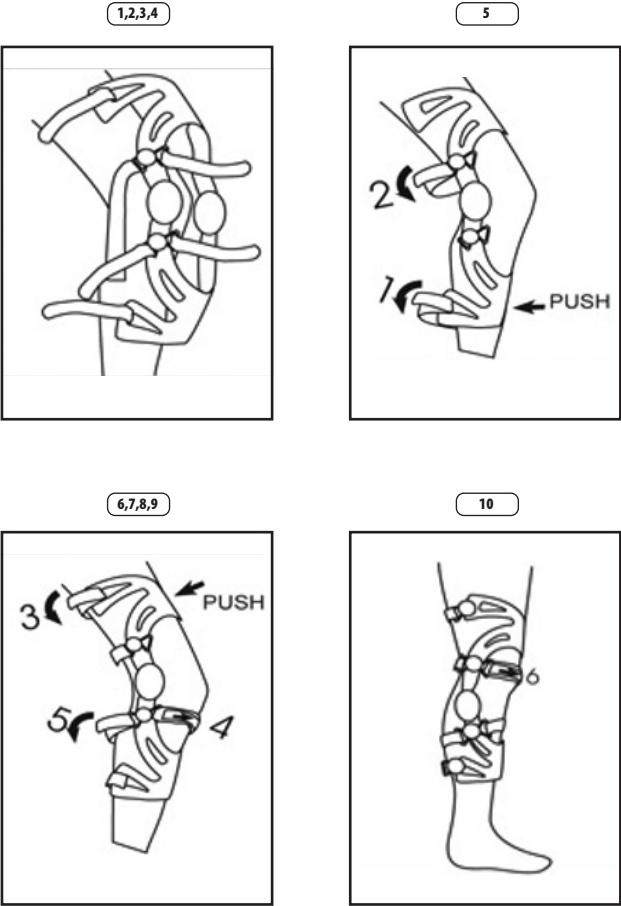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EN

STRIDE SUPPORT FUNCTIONAL KNEE ORTHOSIS WITH A FRAME

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EN

Instructions for use:

The functional knee orthosis effectively stabilises the knee and protects its ligaments after injury, surgery and when returning to sporting and professional activities.

Indications:

- post-traumatic reduction in the function of the musculoskeletal system stabilising the knee,
- status post ACL, PCL, MCL, LCL reconstruction,
- ACL and PCL insufficiency,
- quadriceps weakness,
- return to sporting activity after knee injuries.

Contraindications:

- conditions requiring complete immobilisation of the knee, wounds that are difficult to heal or skin lesions in the area of the orthosis.

Device description:

- an anatomical design, lightweight aluminium frame,
- a polycentric locking system enabling dual restriction of the angular range of motion in flexion (0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°) and extension (0°, 10°, 20°, 30°, 40°) and immobilisation at 0°, 10°, 20°, 30°,
- stabilisation straps to ensure the correct positioning and effectiveness of the orthosis,
- silicone pads on the straps to prevent the orthosis from slipping off the leg,
- fastenings enabling quick putting on and taking off the orthosis,
- the orthosis is available for the left and right limb.

Target groups of patients:

The orthosis is intended for patients with dysfunction of the knee, after injuries, diseases, to support treatment and rehabilitation by providing limitation of flexion and extension of the limb and lateral stabilisation of the knee.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Choosing your orthosis:

1. The first time the orthosis is fitted, it should be done upon prescription and under the supervision of qualified medical personnel.
2. The orthosis should be selected based on the dimensions shown in the size chart.
3. If the device becomes too loose or changes its position, it must be refitted starting from the first step in the section, 'How to put the orthosis on'.

Size chart:

The orthosis is available for the left and right limb.

Size	S	M	L	XL
Thigh circumference [cm]	39–47	47–53	53–60	60–67
Calf circumference (cm)	29–37	37–43	43–50	50–57

Thigh circumference measurement – 15 cm above the centre of the kneecap

Calf circumference measurement – 13 cm below the centre of the kneecap