Motive Plus

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Edition LM09.2012

ASSINGMENT

Motive Plus is a long wrap-around brace of multifunctional, therapeutic use. It effectively supports treatment and rehabilitation for injuries and dysfunctions around the knee joint, in cases not requiring immobilisation.

- Repetitive, subacute pain related to strain injury and micro-injury of muscles and ligaments
- Gonarthrosis and knee joint instability,
- Stabilization after injury during the final phase of functional treatment
- Injury and dysfunction which do not require knee
- Immobilization (lateral ligament injuries).

CONTRAINDICATIONS

- Conditions which require cast immobilization
- Long-healing wounds and ulcers
- Significant bone deformations (brace cannot be adjusted to the joint)
- Lower extremity varicose vein

PRODUCT DESCRIPTION

It is made of advanced, breathable AirX knitted multidirectional elastic fabric, the orthosis ensures suitable comfort of use. Equipped with patellar opening and covered popliteal fossa. Polycentric hinges with stays provide lateral stabilisation. Additional, vertical straps improve stabilization.

PRODUCT ADJUSTMENT

1. Unfasten the brace.
2. Put it on the lower limb, so as to place the patella in the contoured orifice.
3. Fasten The straps.
4. Tighten the straps adjusting the stabilizing force to individual needs.

Size	1	2	3	4	5
Thigh circumference (cm)	36,0 – 39,0	40,0 – 43,0	44,0 – 48,0	49,0 – 54,0	55,0 – 60,0

Remove the stays before washing.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS
TO A MEDICAL SPECIALIST.

mdh sp.z o.o.

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tel: (0048) 42 212 32 08 fax: (0048) 42 674 04 99

www.mdh.pl <http://www.qmedinfo.pl>