PL

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EN

Orthopaedic vest- a modern method of upper limb immobilization of wide therapeutic use. It helps rehabilitation and treatment of contusions and injuries of shoulder, shoulder girdle, arm and elbow joints. The vest allows to eliminate classic methods of upper limb immobilization and related to them complications. It enables effective rehabilitation and high comfort of use.

Indications:

Treatment, follow-up treatment of contusions and injuries of shoulder, shoulder girdle, arm and elbow joints. Follow-up treatment of shoulder girdle and upper limb bone structures. Inflammation and degenerative changes which require immobilization and relief. Post-surgery protection: orthopaedics/ neuroorthopaedics. Prophylaxis and treatment of shoulder dislocation in case of muscular incapacity and soft joint structures (geriatrics).

Product description:

The vest is made of polyamide fabric covered with polyurethane foam.

Additional belt allows to regulate the angle of elbow bend. The vest eliminates necessity of using traditional bandages/ elastic braces, stiff slings etc and minimalises the period of plaster dressing usage. It ensures immobilization and relief of the lower limb. The construction enables adjusting the vest to individual needs. It ensures efficient immobilization and relief of joint structure of upper limb. The vest provides arm adduction, internal rotation in the shoulder joint and forearm and elbow joint abduction. It also allows to regulate the elbow bend within the 90-120 degree range.

Putting on:

The brace should be used according to the instructions of the doctor or physiotherapist.

First Time should be set up by qualified medical personnel who will provide information for further therapeutic intervention.

Maintenance:

Clean the frame elements with most cloth. Padding should be Washed by hand using gentle detergents in temperature max 35°C. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial warmth source. Do not iron.

ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to sweating. Please, do not use gels or warming ointment.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

| Typ | R00 | R0 | R1 | R2 |
|---------------------------|------------------|--------------------|---------------------|----------------------|
| Size | Children (1,5-2) | Children (aged <3) | Children (aged 3-9) | Children (aged 9-14) |
| Thorax circumference (cm) | <50 | 50-60 | 60-70 | 70-80 |

| Typ | R3 | R4 | R5 |
|---------------------------|---------------|---------------|---------------|
| Size | Small; adults | Medium; adult | Large; adults |
| Thorax circumference (cm) | 80-90 | 90-120 | 120< |

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