Disposal of the medical device:

Detach the rigid stabilising elements before washing. Padding and tightening straps should be hand washed at low temperatures (30°) with mild detergents, without softeners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry. Keep away from artificial sources of heat. Check the condition of the orthosis after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

Composition:

PP, PU, PE, POM

Intended user:

The medical device is intended for both laypersons (non-professional users) and professional users.

ATTENTION!

Do not use the device if you have circulatory disorders.

Do not use the device if it has been damaged or deformed.

Do not attempt to repair the device yourself.

Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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PL

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EN

ARMSHELL ARM AND SHOULDER ORTHOSIS

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**Indications:**

Post-operative treatment of fractures of the head of the humerus, glenoid cavity, shoulder dislocations. Degenerative shoulder changes and associated pain. Habitual shoulder dislocations. Recovery after glenohumeral capsular reconstruction. Protection against shoulder dislocation in upper limb paralysis of various aetiologies.

Contraindications:

Conditions requiring immobilisation of the shoulder. Wounds and skin lesions within the area of the orthosis.

Device description:

- the straps ensure stabilisation of the shoulder and the acromioclavicular joint and allow for individual adjustment of the orthosis,
- flexible plastic shell for all-round protection,
- a buckle with a quick-release mechanism for easy fitting,
- universal size for left and right arm.

Target groups of patients:

The orthosis is intended for patients requiring stabilisation of the shoulder area.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Size chart:

Universal size
Biceps circumference: 20 – 41 cm

Size chart:

1. Unfasten all fasteners and buckles.
2. Fasten the straps at the shoulder.
3. Place the strap under the other armpit and fasten it.
4. Adjust the strap lengths to ensure comfort.