Instructions for cleaning:

Padding and tightening straps should be hand-washed at low temperatures (30°) with mild detergents, without softeners. Do not use bleach or strong detergents. Do not spin, do not wring, gently squeeze before drying. To dry, spread the parts of the upholstery on a towel and air-dry. Keep away from artificial sources of heat. Check the condition of the orthosis after each wash. Clean the product systematically, at least twice a month, and more often if necessary.

Composition:

PU, PA, POM, PE, cotton

Intended user:


The medical device is intended for both laypersons (non-professional users) and professional users.

ATTENTION!

Do not use the device if you have circulatory disorders.
Do not use the device if it has been damaged or deformed.
Do not attempt to repair the device yourself.
Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

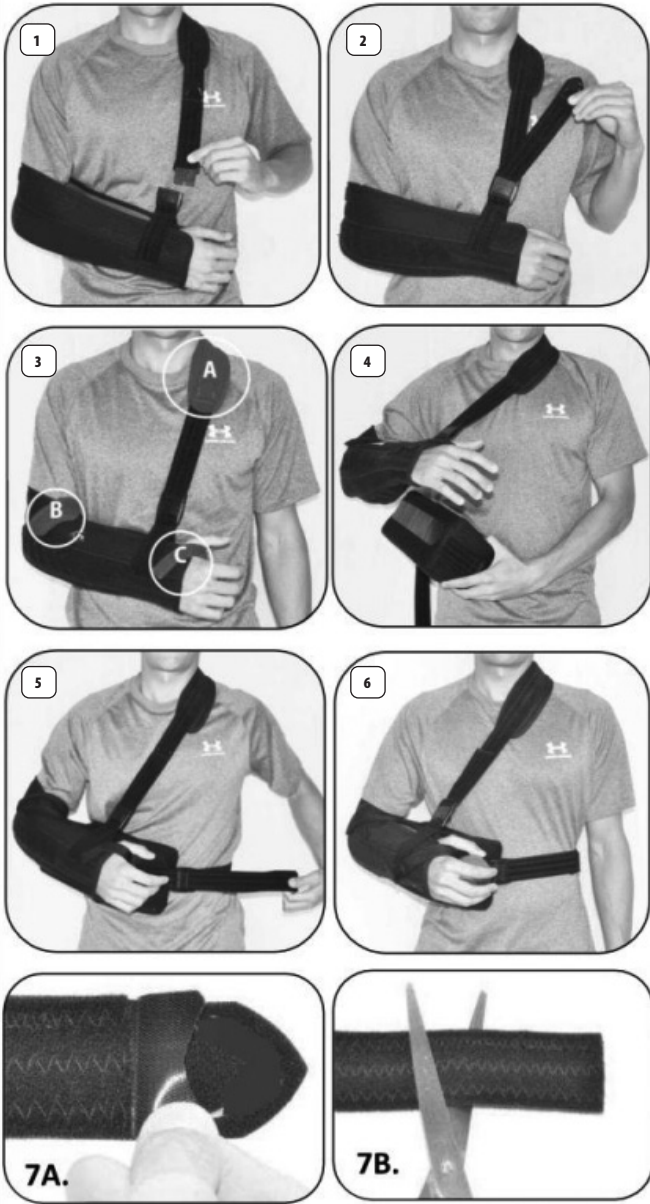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**Instructions for use:**

A shoulder abduction orthosis with wide therapeutic use. Effectively supports treatment and rehabilitation of injuries and dysfunctions of the shoulder, for conditions requiring immobilisation in abduction.

Indications:

Stabilisation of the upper limb in abduction at the shoulder after trauma (sprains, dislocations) or surgery. Treatment after rotator cone reconstruction Immobilisation and unloading of the upper limb in case of inflammatory changes in the shoulder joint.

Contraindications:

Pain and dysfunction of the cervical spine. Conditions requiring immobilisation of the upper limb in another therapeutic position.

Device description:

- the orthosis is made of soft velour,
- immobilisation of the shoulder at a 45° abduction,
- a comfortable foam padding provides more support,
- a buckle with a quick-release mechanism for easy fitting and removal of the orthosis,
- additional exercise ball facilitates hand motion and stimulates circulation.

Target groups of patients:

The orthosis is intended for patients requiring stabilisation of the shoulder area.

Disposal of the medical device:

The device must be disposed of following the applicable local environmental regulations.

Size chart:

Size	S	M	L	XL
Forearm length (cm)	25–28	28–30	30–33.	33–36

Selection of the size of the orthosis is made by measuring the length of the forearm from the olecranon process to the ulnar styloid process.

How to put the orthosis on:

1. Unfasten the buckle and then slide the forearm into the sling. Wrap the adjustable shoulder strap around your neck and shoulders. Fasten the buckle.
2. The shoulder strap has two adjustable ends. Adjust the strap to the correct length.
3. 3A. The shoulder pad is adjustable.
3B. Attach the strap near the elbow.
3C. Attach the thumb strap.
4. Attach the abduction padding to the inside of the orthosis.
5. Wrap the padding strap around your waist and fasten the buckle. Tighten the waist strap and secure.
6. Attach the exercise ball to the end of the padding.
7. 7A. The straps can be shortened if they are too long. Remove the clasp from the end of the strap.
7B. Trim to desired length using scissors. Reattach the clasp.