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INSTRUCTIONS FOR USE

WHOSP - Splint for palm and forearm with thumb spica

PRODUCT DESCRIPTION:

A splint for hand and arm. It has wide range of therapeutic uses, is suitable for patients with orthopedic and neurological disorders.

PRODUCT CHARACTERISTICS:

Wrist and forearm splint with thumb spica, adjustable to patient's needs and progress, light structure made of steel and aluminium covered with PVE foam, removable, soft padding with fingers separator, left and right version.

INDICATIONS:

- Wrist, hand and finger affections requiring support and stabilisation of the affected area in specified corrective and/or pain reducing settings;
- sinew inflammation;
- carpal tunnel syndrome;
- rheumatic joint inflammation;
- contractures preventive treatment;
- circumferential nerves injuries;
- surgery.

USE:

Before the first use, always consult your doctor or physiotherapist who should choose corrective position for the hand. Put on the brace - it should cover the hand and forearm. Fasten stabilization straps.

Size	S	M	L
Wrist circumference (cm)	6,5 – 8,0	8,0 – 9,5	9,5 <

* Size based on hand measurement at the height of metacarpophalangeal joints.

CONSERVATION AND MAINTENANCE:

Product hygiene is important for patient's rehabilitation. The brace should be washed with gentle soap and water. It can be also washed with gentle detergents. Please, do not use the strong chemical detergents and bleaches.

ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric. The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING THE APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A CONSULTANT.