PL

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EN

Use:
KID Air Walking Boot provides stabilization and protection of foot and ankle joint area. Follow-up treatment in case of stabile fractures of ankle/ instep.

Contraindications:
The aircells must not be overinflated as it may lead to skin irritation. Special attention should be paid in case of diabetic patients and patients with sensory disturbances. In case of skin changes or irritations, the compression should be decreased.
The product cannot be used in case of open, unstable fractures, difficult to heal wounds, sores and other skin changes. It is necessary to contact a doctor in case of pain or numbness.
Special attention should be paid while walking on wet or slippery surfaces. It is necessary to contact a physiotherapist or a doctor before changing the way of using the brace.
The product should be used only under careful medical supervision.

Placement instruction:

- Unfasten straps and remove inner soft liner from the boot.
- Open the soft liner and place foot inside with the heel fitting snugly into the back portion of the liner. Fasten and wrap foot flaps then the leg portion of the liner. Ensure liner is snug but not constrictive. Make sure pump is facing outward.
- Gently spread the side struts and place the lower leg into the brace.
- The ankle should be centered at the struts with the foot and ankle at 90°.
- Fasten the Velcro straps beginning at the toe then moving up the leg. Do not over tighten. As swelling pattern changes throughout the day*, it may be necessary to adjust the straps for comfort.
*Due to activites, rest, etc.
- INFLATE
To inflate air liners, press and release center air bulb until liner provides a comfortable cushion and support. Do NOT over inflate.
DEFLATE
To reduce the pressure in the orthosis chambers, turn the valve on the front of the orthosis.
- To remove, simply unfasten the straps without removing them completely from the assembled walker. Unfasten the soft good liner and gently bring the foot and ankle forward and out of the walker. To reapply, place the foot and ankle into the liner and reattach the Velcro closures

Washing instructions:
Remove foam liner for hand wash, using a mild detergent and cold water, air dry. Do not wash in washing machine or in mechanical dryer. For plastic components, clean with a damp cloth, air dry. Do not use detergent or bleach.

Maintenance:
The inner liner should be washed using sponge and mild detergent. (but is should not soak through). Dry far away from artificial source of heat. Dry in a flat position. Do not iron. The cover should be cleaned with a moist cloth.

CAUTION

- If discomfort or pain results, persists or increases, discontinue to use and consult with medical professional.
- Make sure the hook and loop fasteners are well fastened while wearing the product. If the hook and loop fasteners can not be Fastened well, stop using the product to prevent injuries.

THE FINAL DECISION CONCERNING THE USAGE AND SELECTION OF THE APPLIANCE SHALL BE MADE BY THE DOCTOR SPECIALIZING IN THE FIELD

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