
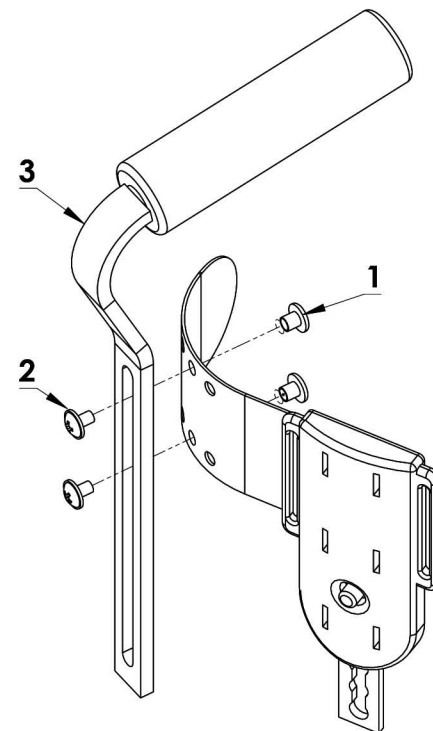
Producent/Manufacturer:

mdh sp. z o.o.
ul. Tymienieckiego 22/24
90-349 Łódź
Tel. +4842 637 59 79
Fax +4842 636 52 21
www.mdh.pl

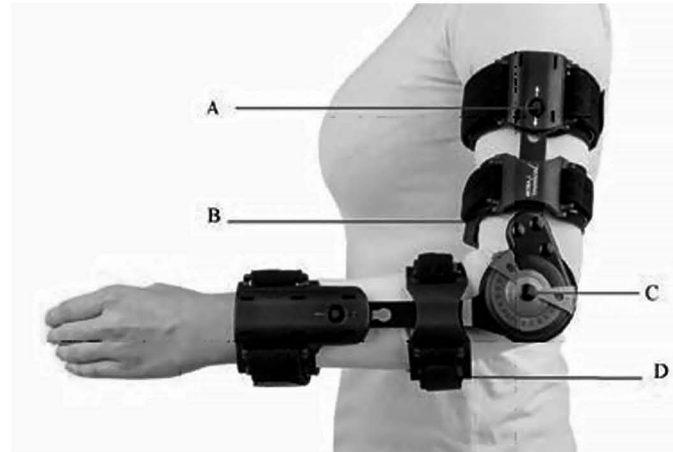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Indication:

Treatment, follow-up treatment and rehabilitation of joint injuries which require the limitation of the pronation/supination motions in the forearm with the motion range control in the elbow joint.

Contraindications:

Conditions which require the immobilization of the elbow joint using a plaster cast. Difficult-to-heal wounds or skin lesions within the brace coverage area. A particular care needs to be taken in patients with sensory disorders.

NOTE:

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the brace should be put on by experienced health care personnel. The angular range of motion should be adjusted according to instructions given by the doctor or physiotherapist. Before use, please read the user manual carefully. If you feel discomfort while using the brace, there is swelling or skin irritation, please consult your doctor or physiotherapist.

Photograph legend:

- A A button for the infinite adjustment of the brace length
- B Fabric pads preventing the brace from slipping or rotating (anti-slip system)
- C A hinge for adjusting the angular range of motion or immobilising the elbow joint
- D A flexible cuff to match the brace to the limb circumference

Fitting instruction:

- Brace length adjustment: Place the brace in a way that the hinge axis is right over the lateral epicondyle (outer side of the elbow). In order to adjust the length, press the button and slide the arm part to achieve the desired length. The brace should provide maximum comfort of use and proper stabilisation (it should extend up to the axillary fossa). The forearm part should be adjusted in the same way. Its length should provide appropriate stabilisation (the brace should extend up to the ulnar styloid process — the outer edge of the wrist).
- If the angular range of motion needs to be adjusted, it should be done before putting the brace on.
- In order to set the selected angular range of motion, press the lock and slide it so that the number that represents the selected angle is in the opening. The same steps should be taken to set the flexion and extension ranges. In order to fix the selected angular range of motion, the plastic clamp (supplied with the brace) should be inserted through the opening in the flexion and/or extension lock.
- In order to set the appropriate circumference of the brace, fasten the stabilising straps, starting with the straps closest to the hinge. The plastic cuffs need to be matched to the limb circumference and shape. If necessary, the stabilising straps need to be trimmed and the hook and loop fastener located on their ends need to be removed to the new, trimmed strap end.
- In order to take off the brace, undo all the stabilising straps and loosen the cuffs.

Support strap application:

- Clip the support strap fixing ring to the wrist strap.
- Put the strap on the other, uninjured limb, move it behind your back and under the braced limb.
- Fasten the end of the strap to the fixing ring using the bracket located on its end.
- The strap length needs to be adjusted in a way that the limb is in a comfortable position by undoing the hook and loop fastener located on the front of the strap.

Holder fixing (Elbow Pro Plus):

- Insert the two threaded nuts (1) into the opening that fits to the forearm so that it is aligned with the hand holder.
- Apply the holder with a grip (3) onto the nuts and fasten both bolts (2) to the initial signs of resistance.
- Adjust the length, by moving the holder (3) along the clearing, then tighten the bolt (2) to lock the holder in the position.

Maintenance

The straps and the fabric pads should be hand-washed using mild detergents. Do not chlorinate. Do not machine wash. Dry away from artificial heat sources.

NOTE:

A failure to observe personal hygiene and using inappropriate detergents may result in skin abrasion or other symptoms arising from excessive perspiration and bacterial growth. THE FINAL DECISION REGARDING THE USE AND CHOICE OF EQUIPMENT MUST BE TAKEN BY A PHYSICIAN

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