**INDICATIONS:**

Conditions requiring the immobilization of the thoracic and cervical spine. Post-operative stabilization.

**CONTRAINDICATIONS:**

None. The attending physician decides whether or not to use the product.

**PRODUCT DESCRIPTION:**

The set consists of mandible and occiput supports which need to be adapted to the ONE orthopaedic corset with V-shaped back pelotte using the screws included with the set. The product is finally fitted to an individual patient using leather straps.

**USE:**

1. Set the appropriate height of the supports.
2. Using 2 screws, mount the occiput support to the V-shaped back pelotte, while the mandible support should be mounted with 2 screws to the sternum pelotte.
3. Using the straps, connect the occiput support with the mandible support to ensure head stabilization.

**MAINTENANCE:**

The lining elements can be hand washed in very mild washing agents at no more than 35°C, do not chlorinate, do not dry wash, do not wring, do not spin dry; squeeze gently before drying. Dry away from artificial heat sources. Spread to dry. Do not iron.

**NOTE!**

Improper personal hygiene or using unsuitable washing agents may cause abrasions or different symptoms arising from excessive perspiration and bacterial growth. Remember that unrinsed detergent residues can irritate the skin and damage the product material.

Pay particular attention to the skin condition within the area of contact between the supports and the patient skin. If any lesions occur, such as redness or irritation, immediately consult qualified medical personnel.

THE PRODUCT IS NOT FIREPROOF. DO NOT USE IN DIRECT PROXIMITY TO OPEN FIRE.

THE FINAL DECISION REGARDING THE APPLICATION AND SELECTION OF EQUIPMENT IS MADE BY A SPECIALIST PHYSICIAN

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