### Composition:

PU, PE, PA, PET

### Intended user:

The medical device is intended for both laypersons (non-professional users) and professional users.

### ATTENTION!

Do not use the device if you have circulatory disorders.

Do not use the device if it has been damaged or deformed.

Do not attempt to repair the device yourself.

Do not use the device of the wrong size or in a way for which it is not intended. If you experience any problems related to using the product, please get in touch with your doctor. Failure to observe proper personal hygiene and inappropriate laundry detergents can cause abrasions or other ailments associated with increased sweating and bacterial growth. Do not use the orthosis with warming gels or ointments. Any serious incident in connection with a device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is based.

IT IS RECOMMENDED TO CONSULT A SPECIALIST BEFORE USING THE DEVICE

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**Intended use/Indications:**

Conditions of the neck area that do not require absolute full immobilisation, muscle and ligament injuries, overload and inflammation of the musculoskeletal system, intervertebral joint dysfunction with or without trauma, intervertebral disc damage, arthroses, rheumatoid diseases, fracture healing.

**Contraindications:**

Severe pain in spinal deforming lesions and discopathies, post-operative conditions, fractures and dislocations of the cervical vertebrae.

**Device description:**

A lightweight, one-piece, anatomically shaped, polyurethane foam collar with an adapted density. Velcro fastening and circumference adjustment. The collar trim has antibacterial and antifungal properties. The stiffening increases the stabilising effect.

**Target groups of patients:**

The orthosis is intended for patients with conditions of the neck area which do not require absolute and full immobilisation.

**Disposal of the medical device:**

The device must be disposed of following the applicable local environmental regulations.

**Choosing your orthosis:**

The correct use of the collar is as important as the correct sizing for correct immobilisation and comfort. Use Velcro to moderately tighten and fasten the collar.

**Size chart:**

Size	Neck circumference (cm)	Collar height (cm)
XS	25-32	6.4
S	32-38	7.6
M	38-46	9.9
L	46-51	10.2
XL	51-56	11.4

**Instructions for cleaning:**

The collar can be washed by hand in lukewarm water with mild detergents. Do not machine-wash or chlorinate. Do not wring out, squeeze gently. Dry unfolded away from artificial heat sources.