FUNCTIONING:

- Reducing the amplitude related to bending-unbending the spine in LS area.
- Equalizing the pression on intervertebral disc and intervertebral joints.
- Supporting and relieving the weakened and tense muscles.
- Stabilization of L-S spine.

USE:

Do not forget to make sure that the brace size has been chosen carefully, that is essential for proper functioning of the corset. In order to put on the brace, the patient should lie down. Put the brace so that it was placed 2 cm above the coccygeal vertebrae and the brace's label (you can find it inside the brace) it is directed upward. Back liners should adhere to lumbar-sacral area. Fasten the belts and regulate the dorsal pressure. Make sure that the pressure provides optimal stabilization but do not disturb breathing. Time of using the brace should be precised by medical specialist and should be accompanied by rehabilitation exercises program.

Size	1	2	3	4	5	6
Waist circumference (cm)	52 - 62	63 - 74	75 - 89	90 - 105	106 - 120	121 - 140
Height (cm)	31					

CONSERVATION AND MAINTENANCE:

The brace should be washed with gentle detergents in max. temperature 35°. Do not chlorinate, do not dry clean, wring gently before drying. Do not dry close to the artificial heat sources. Dry in horizontal position. Do not iron.

ATTENTION!

Not observing the rules of personal hygiene and cleaning the vest with inappropriate detergents may result in skin abrasion or other effects related to perspiration. Please, do not use gels or warming ointment. Always rinse carefully after washing-detergent residue can cause skin irritation and damage product material.

The product is not fireproof. Do not use close to the open sources of fire.

THE FINAL DECISION CONCERNING APPLICATION AND CHOICE OF THE EQUIPMENT BELONGS TO A MEDICAL SPECIALIST.

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INSTRUCTIONS FOR USE

SPINCARE FRAME – Lumbosacral orthosis

Spincare Frame corset effectively helps rehabilitation and treatment in case of post-traumatic injury, diseases which disturb effectiveness of osseous structures or ligaments located in lumbar area. It restrains spine movement range in intervertebral joints, enables effective stabilization, relief and correction of body posture which lead to soothing clinical symptoms connected to degenerative changes, inflammation. Increasing the pressure in abdominal cavity, it helps to reduce axial pression in spine regions affected by pathological changes. It is also useful as prophylactic aid.

INDICATIONS:

Intense and chronic pain syndromes of lumbar-sacral spine area; discopathy; degenerative changes of lumbar sacral spine area; stabilization after surgery and during functional treatment; instability of lumbosacral area.

PRODUCT DESCRIPTION:

Flexible fabric carries humidity and warmth away. In the dorsal part of the orthosis, there are four tunnels, parallel to the centre line of the body, equipped with stays which provide passive front and rear trunk stabilization. Special belt loops facilitate putting on and off.